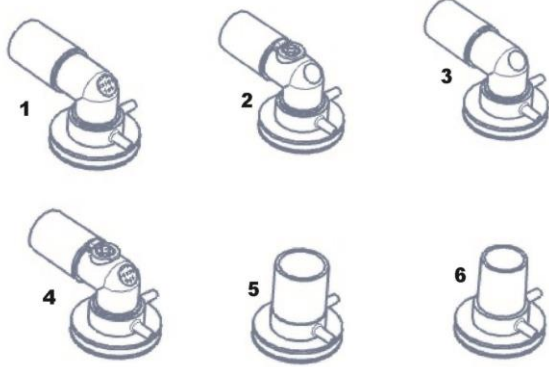
TR

1984

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Yayın Tarihi: 09.06.2016

## GECE ORA-NASAL MASK USER MANUAL

Mask air inlet port should be chosen appropriately to the device that the patient use.

### 1. Usage Method

1. With exhalation port
2. Without exhalation port AAV\*
3. Without exhalation port
4. With exhalation port AAV\*
5. Straight port 22mm F
6. Straight port 22mm M

The six-different configuration below are presented for the users.

The ports without exhalation valve are for ventilators,

The ports with exhalation are for the devices that need air outlet valve like CPAP, BiPAP, BiPAP ST.

\*AAV; Anti-Asphyxia Valve

This mask is designed to be used by the patients, who faces respiratory disorders, with the devices that have adequate alarms and safety systems. The mask can be used by multi-patients suitable for noninvasive ventilation at home, hospital or other clinical environments.

### 2. Cautions:

The mask should be sold or prescribed by a physician. It also should be used with a supervision of a physician. In order to use the mask effectively, patients with facial hair should be shaved before use.

### 3. Warnings:

- a. You should pay attention that while using the mask without AAV, the patient cannot breathe in room oxygen if the ventilator or respiration device breaks down or stops.
- b. In case of a breakdown of the ventilator, this mask requires the same attention and care as invasive use.
- c. This mask can only be used with devices that have adequate alarms and safety systems.
- d. This mask should not be used by patients who are uncooperative, obtunded, unresponsive or unable to remove the mask.
- e. If you face any unusual chest discomfort, shortness of breath, stomach distention, belching or severe headache, inform your physician immediately.
- f. To minimize the risk of vomiting avoid eating and drinking for three hours before using the mask.
- g. If you face skin irritation, inform your physician.
- h. In order to prevent the risk of infection between patients in ventilators such as rental units you should use bacterial filter in patient circuits.
- i. If a fixed flow rate of oxygen is used, the percentage of the inhaled oxygen will vary depending on the pressure settings, breathing pattern of the patient, mask size and leakage.
- j. The masks are sold clean but nonsterile. If it is necessary to be sterilized, then follow the steps of Sterilization in User Manual.
- k. As the mask will not stay sterile in repeated use or multi patient use you should follow the steps of sterilization in User Manual.
- l. Although Foras Medical determined the sterilization method, sterilization institution may use its own method.
- m. Do not use the mask on open wounds.
- n. In order to prevent the risk of infection, change the mask with the new one when it becomes unclean able.
- o. Stop using the mask in cases of skin irritation and allergic reaction.
- p. Do not use the mask when the device is not connected or operating properly.
- r. The mask should be checked regularly for the flow blockage and restrictions.
- s. In order to remove the mask quickly in emergency, the connections should be in appropriate position that the patient can remove.

t. In order to prevent the severe hazards, the device should have low pressure, connection failure, power loss and air supply loss alarms.

#### **4. Contraindications:**

The mask will not remain sterile even single patient uses it repeatedly. In order to prevent infection do not use on open wounds. The methods of cleaning, disinfection and sterilization are available in User Manual. Foras Gece masks may not be suitable for those patients:

1. unconsciousness
2. unable to remove the mask
3. open wounds that has risk of infection
4. hemodynamic or cardiorespiratory instability
5. claustrophobia, anxiety or discomfort about Oro-nasal Mask
6. facial or nasal deformity, beard or some other reasons to not to use the mask properly
7. excessive reflux and disorders about secretions
8. continuous cough, hiatal hernia, inability to swallow or clear secretions
9. upper airway obstruction or facial trauma
10. barotrauma
11. recent facial or gastric surgery
12. patients using drugs that may cause them vomit
13. patients' needs immediate intubation

#### **5. Complications:**

Foras Gece Masks are non-invasive products. The silicone surface which is applied directly to the patient's skin is made up of soft, pliable and biocompatible material.

Although the possibility is so low; some minor complications are:

1. infection on open wounds as a result of improper use
2. skin irritation after using the mask long periods
3. dental or nasal pain, deformity
4. throat or nasal drying
5. eye irritation or conjunctivitis
6. gastric distention, stomachache or flatulence
7. after long period usage, some minor discomfort
8. decreased secretion clearance
9. aspiration of secretions

#### **6. Mask components and Material Description**

- a. Face piece is made up of silicone. It has different semi-transparent colors according to the sizes. Its name and size are on the mask labelled on molding. There is a place for the swivel port and there are four integrated ports for the headgear clips.
- b. Swivel port can rotate 360 degrees. There are 6 different models with elbow or straight. Plastic parts are made up of semi-transparent polycarbonate and the plugs are silicone.
- c. Headgear has a mechanism that lets the patients remove the mask easily. When you disconnect the clips the mask falls from the face. When connected properly, four clips carry the mask well. Thanks to the Velcro on the stripes you can easily adjust the tightness.

#### **7. Safety Features**

Easily removable clips allow the patients to remove the mask quickly in emergency. With the masks with anti-asphyxia valve the risk of harm of the ventilator is minimized.

#### **8. Applying and Adjusting the Mask**

- a. Mask and Headgear Sizes
  1. L (Large), M (Medium), S (Small), XS (Extra Small), XXS (Extra extra Small)
  2. Headgear sizes: L (Large), S (Small). We recommend L headgear with L, M and S masks. S headgear is suitable for XS and XXS masks.
- b. Although the best method is to place the mask on the patient face there is also Mask sizing scale. For mask sizing the patient's facial muscles should be relaxed and chin closed. Put the chin in to the chin's

section and the top should be below the nasal root.

c. Applying the mask and headgear

1. Extend the length of the four stripes without pulling them to the clips.
2. Slip each of the clips on to the correct face piece nimb. Pay attention that the bottom clips should be the ones easily removable.
3. with mask in one hand and the headgear in another put the mask on to the face of the patient correctly. The top headgear stripes should be located above the ears and below the eyes. And the bottom ones should come below the ears.
4. After placing the mask correctly adjust the stripes paying attention not to be so tight.
5. While adjusting the mask, ask to the patient about his/her comfort.
6. Connect the tube coming from the ventilator to the 22mm swivel port.

#### 9. Removing the Mask and Headgear:

- a. Easily removable mechanism includes 4 two part clips connected to the stripes. These clips let the patients fit and remove the mask without changing the adjustment of tightness each time. To disconnect the clips, squeeze the sides of the clip with two fingers and pull back. To connect the clips simply press the male into the female half and it will snap in place.
- b. Storage: When not using the masks, keep them in a place preventing contamination and damage.

#### 10. Cleaning, Disinfection and Sterilization:

It is recommended that the one handling the contaminated devices wear gloves.

The mask main body and the silicone headgear:

Autoclave: 4 minutes at 132 °C degree

Ethylene Oxide: 55 °C degree

The headgear made up of cloth cannot be sterilized.

#### Cleaning

Submerge all the components 5 minutes in mild water with detergent. Scrub the submerged mask with a soft bristled brush. Agitate the mask in the solution while scrubbing. You can use small cytology type brush for internal parts. Rinse the mask with 38-49 °C degree tap water. Then, rinse again for one minute. Apply these two times. Dry the mask with a clean, lint free cloth.

#### Cleaning the headgear

You can clean the silicone headgear in the same way as the mask.

For the cloth headgear;

Submerge the headgear in soapy warm water (38/49 °C) and gently rub all areas. Rinse the headgear in clean water for one minute and let air dry.

#### Cautions:

The headgear cannot be sterilized and cannot be used on multiple patients.

In first a few cleanings the material may run black dye.

After each process check all the parts of the mask. If there is a damage on any, change that part with the new one. Do not use bleach, chlorine or alcohol based solutions in the cleaning of any parts.

It is the user's responsibility to validate any deviations from these recommended methods.

Before using the mask with another patient always sterilize or disinfect it.

#### 11. Recommendations for Mask Disposal

All the components of this mask are considered non-critical and should be disposed as medical waste in accordance with the local law that is used.

Foras Bilg. Elekt. Medikal San. Tic. Ltd. Şti

Ostim Mahallesi 1466.Sokak No: 28

Yenimahalle - ANKARA

**Made in TURKEY**

[www.foras.com.tr](http://www.foras.com.tr)

[sales@foras.com.tr](mailto:sales@foras.com.tr) / [info@foras.com.tr](mailto:info@foras.com.tr)

FR.07.14.04 Rev 02

Publication Date:

03.05.2011

Discover  
the potential