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Rev 02

Publish Date: 09.06.2016

## SleepAs Ora-Nasal Mask and Gel Mask User Manual

SleepAs Ora-Nasal Mask and Gel Mask are intended to be used for patients in home or hospital under CPAP, AUTOCPAP, BiPAP, BiPAP S/T or non-invasive ventilation therapy.

**Warning:** This Mask is designed for use only after recommended by a health care professional or respiratory therapist.

### Mask Components and Materials:

The package includes one unit of mask main body, one unit of elbow and a headgear. The cushion of the mask is made up of gel and silicone and it is translucent. Plastic main body and the elbows are made up of silicone. They are also translucent. The tap of the port is TPR and the headgear is neoprene.

### Warning

- Accordance with its intended use, it should be decided by and used under the control of a physician as whether to be with an exhalation port, with an anti-asphyxia valve or without any exhalation port.
- Do not use the mask in patients with disorder in cardiac sphincter, cough, acid reflux and hernia.
- Do not use if the patient is unable to move or take off the mask.
- Do not use in conditions such as an inflation in chest, short breaths, belching or headache.
- Stop eating or drinking up to three hours before using the mask in order to prevent vomiting.
- If the patient takes medicine that causes him/her to vomit, do consult to the doctor.
- Do not wear the mask if the CPAP or BiPAP device is not switched on and operating properly.
- Do not block the exhalation ports on the mask.
- Replace the mask or the equipment if you realize any damage or deformation.
- Product is non-sterile.
- Do not use the mask if the package is opened or damaged.

### Cleaning Instructions

- Separate the cap from the mask.
- Do hand wash the mask and equipment in mild water with a liquid dish washing detergent.
- Rinse thoroughly. Before use, pat with a paper towel and air dry in room temperature.

- As they cause the mask to become crusty, do not use bleach, alcohol or cleaning solutions containing alcohol, conditioner or moisturizers.
- To make the patient being comfort clean the mask before each use.

**Connecting the Mask to other Medical Devices:**

Connecting the mask to a humidifier, nebulizer, filter, port or extra oxygen supplier does not affect the mask.

**Mask Life and Disposal**

The service life of the mask is 5 years. All parts of the mask are accepted as noncritical material. They can be disposed in accordance with the laws of the region and medical waste method.

**Technical Specifications:**

The Leakage rates at specific pressure are below:

10 cm H<sub>2</sub>O: 30 L/min

20 cm H<sub>2</sub>O: 41 L/min

30 cm H<sub>2</sub>O: 49 L/min

The pressure at specific flows are below:

50 L/min : 0,5 cm H<sub>2</sub>O

100 L/min: 1 cm H<sub>2</sub>O

 Refer to the document given together	 1984 CE mark and notified body no.	 Read the user manual	 Product is non-sterile	 Do not use if package damaged
 Do not expose to sunlight	 Storage temperature limits	 Storage humidity limits	 No latex	 Manufacturer
 Product Lot No	 Manufacture date	 Expire date		

Foras Bilg. Elek. Medikal San. ve Tic. Ltd. Şti.

**Made in TURKEY**

Discover  
the potential

Ostim Mah. 1466. Sok. No.28 06370

Yenimahalle / ANKARA - TURKEY

Tel: +90 312 395 77 66

Fax: +90 312 395 77 67

[www.foras.com.tr](http://www.foras.com.tr) / [sales@foras.com.tr](mailto:sales@foras.com.tr) / [info@foras.com.tr](mailto:info@foras.com.tr)

FR.07.14.02 Rev 02

Publish Date: 09.06.2016