

**DİSPOSABLE CHAMBER KULLANIM KILAVUZU**  
(SN-260, SN-560, SN-300, SN-300 A/F, SN-400, SN-400 A/F)

<b>KULLANIM KILAVUZU</b> <b>DİKKAT EDİLECEK HUSUSLAR</b>	<b>USER MANUAL</b> <b>PAY ATTENTION</b>
<ul style="list-style-type: none"><li>* Hava Giriş ve Çıkışları 22mm</li><li>* Kullanıldığı humidifier üzerinde insana zarar verebilecek ısı olabilir, el ile dokunulmamalıdır.</li><li>* Herhangi bir yerinde su sızdırdığı gözlenirse yenisi ile değiştirilmelidir.</li><li>* İçerisine distile su konulmalıdır.</li><li>* Üzerinden sızdırma olup olmadığı periyodik olarak gözlemlenmelidir.</li><li>* Tek kullanımlık olarak üretilmiştir otoklav yapılmaz.</li><li>* İçerisine konulan su, su seviyesini geçmemelidir.</li><li>* İçerisine Sıcak su konulmaz.</li><li>* Raf ömrü tarihinden itibaren 2 yıldır.</li><li>* Kullanım ömrü tek hastada 15 gündür.</li><li>* Üretim tarihi ve lot numarası ambalaj üzerinde yazmaktadır.</li><li>* Hortumlardan su gitme riskine karşı, düz zeminde çalıştırılmasına dikkat edilmelidir.</li></ul> <p><b>Otomatik Su Doldurmalı Modellerde;</b></p> <ul style="list-style-type: none"><li>* Otomatik su doldurma hortumunun kırılmamasına dikkat edilmelidir.</li><li>* Damla sistemindeki filtrenin tıkanmasından dolayı su akışı kesilirse, suyun kalitesi kontrol edilmeli ve chamber değiştirilmelidir.</li><li>* Beraberinde verilen damla seti ve diğer malzemelerin haricinde malzeme ile kullanılmaz.</li><li>* Şamandıradan oluşacak hatadan dolayı su doldurma seviyesinden fazla su dolar ise su hortumu üzerindeki manuel kapatma ile hortumdan akan su kapatılır.</li><li>* Otomatik su doldurmalı chamberların şamandıra seviyesi, yüksek performansta olabilmesi için en uygun su seviyesine göre ayarlanmıştır.</li></ul>	<ul style="list-style-type: none"><li>* Inlet and outlet air port, 22mm</li><li>* There may be high level of temperature on the humidifier which might harm to person. So do not touch with hand.</li><li>* If any leaking of water on the the chamber is observed change it with the new chamber</li><li>* The water of the chamber should be distilled.</li><li>* Do not use tap water in chamber</li><li>* It should be monitored periodically if there is any leaking on the chamber.</li><li>* Do not reuse the chamber they are manufactured for single use. They can not be autoclaved.</li><li>* The water should not exceed the water level of the chamber</li><li>* Do the use hot water</li><li>* Shelf life is 2 year dating from the production date</li><li>* Life expectancy is 15 days in a single patient.</li><li>* The production date and lot number are written on the package.</li><li>* Make sure that it works on a straight surface to avoid the risk of water delivery to the patient from the hoses.</li></ul> <p><b>For Automatic Fill Chambers;</b></p> <ul style="list-style-type: none"><li>* Attention should be paid for automatic water supply hose not to be cracked or broken</li><li>* In cases where the water supply is discontinued due to clogging of the drop system filter, the water quality should be checked, and the chamber be replaced</li><li>* It may not be used with any material other than the drop set any other materials delivered with the unit</li><li>* In case of filling higher than the water fill level as a result of any possible failure or defect in the float the manual shut off valve on the water hose should be used to stop the water flow from the hose</li><li>* To make to automatic water filling chamber operate in high performance, the float has been fixed according to the appropriate water level .</li></ul>
<p> <b>For As Bilg. Elekt. Medikal San. ve Tic. Ltd. Şti</b> Ostim Mahallesi 1466. Sokak (Eski 677. Sokak) No : 28 Yenimahalle / ANKARA / TÜRKİYE</p> <p>Tel: (+90 312) 395 77 66 Fax: (+90 312) 395 77 67 <a href="mailto:info@foras.com.tr">info@foras.com.tr</a> <a href="http://www.foras.com.tr">www.foras.com.tr</a> <b>MADE IN TURKEY</b></p> <p>FR.07.12.03 Rev. 05 Yayın Tarihi: 10.06.2013</p>	<p style="text-align: center;">   </p>